



## Apply


1. Using both hands, place two fingers inside the **CT comp.** pulling on each side with equal force.
2. Insert a finger between the stocking and the foot. Gently pull the **CT comp.** over the ankle.
3. Use the thumb and forefinger while pulling it up in small increments, smoothing it as you go.

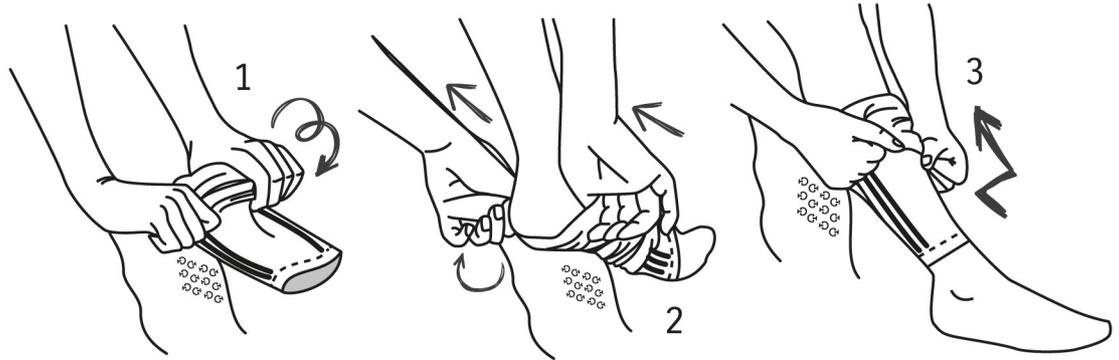

## Remove

1. Grasp the top of the **CT comp.** and begin turning it inside out; gently pull down to the ankle.
2. Insert a finger between the stocking and the leg, gently pull the **CT comp.** off the ankle. Slowly pull out the foot.

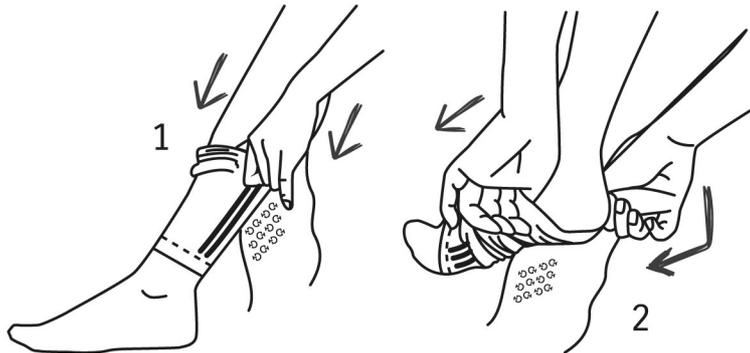

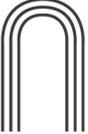  
**COMPRESSION**  
ENJOY THE REVOLUTION

  
CAVALLERIA  
TOSCANA